## GELSCAR - GELTOP - PREVOGEL - OS'CARE

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## GELSCAR - GELTOP - PREVOGEL - OS'CARE

**GELSCAR CUSHION** VISCOFLUID GEL WITH A NEOPRENE CELL COVER  
**GELTOP CUSHION** VISCOFLUID GEL ON TOP OF A HIGH RESILIENCE FOAM  
**W/TH PROTECTED BY A SEMIPERMEABLE COVER WITH ANTI-SLIP BASE**

**PREVOGEL CUSHION** VISCOELASTIC GEL ON TOP OF A HIGH RESILIENCE FOAM IN A NEOPRENE CELL COVER

**OS'CARE CUSHION** VISCOELASTIC GEL

## TIPS

The first line in pressure ulcer prevention involves changing the pressure points of the body on the cushion an average of every two to three hours, to limit prolonged compression of blood vessels in relation to bony protrusions and to allow tissue irrigation.

## INDICATIONS

- For patients with low to average risk of pressure ulcers, evaluated according to a tested scale and clinical judgement.
- Established pressure ulcers: follow medical recommendations.

## CONTRAINDICATIONS – GELSCAR CUSHION

- Users with frontal and sagittal stability problems
- Paralysis of the trunk and/or lower limbs.

## INSTALLATION

The cushion is delivered "ready to install". In the event of changing the cover ensure that the gel surface is always facing upwards and the rubber surface is towards the bottom.

• **GELTOP - OS'CARE** : The non-slip surface is the lower side.

• **GELSCAR** : Tie the 4 strings to the frame of the chair.

## CLEANING / DISINFECTION

• Promust PU, PUHD ,Techmapolar

• Neoprene cover

## SAFETY

- **GELSCAR** : Limit it to users weighing between 50 and 110 kg.
- **GELTOP** : Limit it to users weighing between 40 and 95 kg.
- **OS'CARE** : Limit it to users weighing between 40 and 90 kg.
- **PREVOGEL** : Limit it to users weighing between 50 and 110 kg.

## Precautions for protective coverings

- Systematically protect the cushion by using the supplied fully waterproof covers.
- Prohibit scouring, stripping or dissolving products and sharp objects in direct contact with the protective covering.
- Make sure you regularly inspect the normal wear on the protective covering and replace it if it is damaged (holes, cracks, changes in the surface, transparent when looked at in the light).

## Precautions for materials

- Perform each year, a visual checks as to the fluidity and the aspect of the gel when touched

## STORAGE

- Cushions should preferably be stored flat, away from humidity.

## DISPOSAL

- Do not burn or dump the support and its protective covering outside areas intended for this purpose.

## GUARANTEE

The duration of the legal guarantee for cushions is 2 years.

This guarantee starts on the date of purchasing the product from your distributor.

In the event of single patient usage, the guarantee covers the support and its protective covering. In the case of multi-patient usage, we recommend replacing the protective covers every 2 years. This guarantee does not cover normal wear on the product and its protective covering and does not substitute legal guarantees.

Please contact YOUR DISTRIBUTOR and show him your faulty product.

They will carry out the necessary procedures with our company in order to carry out repairs or make an exchange.

## PRODUCTS LIFETIME :

- Estimated lifetime of GELTOP cushion is 4 years

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