SK

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09/2023 - V3

GB

References	Description
09/8000	Urinal UROLIS
09/8003	Female adapter

Description:

- Depth : 24 cm, Height : 14 cm
- Overall length : 30 cm
- Front width : 8,5 cm , back width : 14 cm
- Net weight Urolis : 330 g, adaptater : 30 g
- Capacity : 1,5 liters

Storage :

Disposal conditions: The medical device must not be discarded in nature and must be disposed of in a recycling center.

Material:

- Container: High Density Polyethylene.
- Cap: Low Density Polyethylene.
- Cone: High Density Polyethylene, elastomer seal, stainless steel ball.
- Draining plug: 30% glass-fiber reinforced polyamide 6, elastomer seal.
- Female adapter : mixture of elastomer and polypropylene homopolymer.

Precautions of use:

- Keep out of reach of children
- WinnCare France can not be held responsible for any damage related to improper use of the product.

Advantages:

- Urolis can be used safely in a sitting or a lying position
- Thanks to its spill-proof safety system, the content of the Urolis cannot spill into the bed
- A female adapter that fits directly on the cone is available
- The use and cleaning of the Urolis are facilitated by its opening and closing systems: turn the ring and the cone can be easily removed
- Draining plug with elastomer seal
- Level graduation
- Anti-odour safety cap
- Ergonomical handle

Cleaning:

- 1 Remove the cap from the handle
 - 2 Remove the female adapter. Turn the ring for easy removal of the tip
 - 3 Remove the draining plug
- Water and usual disinfectant (bleach type)
 - Autoclave 130°

Warranty: This product has a 2 years warranty against any manufacturing defects from the date of purchase. The warranty is only valid if the washing and caring instructions have been followed

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the patient is established

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